



අභ්‍යන්තර සටහන

පළාත් මහ රෝහල - බදුල්ල

දිනය	2023.08.22
වෙත	අදාල සියලුම විශේෂඥ වෛද්‍යවරුන්/ වෛද්‍යවරුන්/කාර්යභාර හෙද නිලධාරීන්
වෙතින්	අධ්‍යක්ෂ
පිටපත්	ආසාදන පාලන ඒකකය
විෂයය	නොමේරු ළදරු ඒකකයේ CX 50 අල්ට්‍රාසවුන්ඩ් පද්ධතිය හා වෛද්‍ය උපාංග පිරිසිදු කිරීමේ ක්‍රමවේදය පිළිබඳව දැනුවත් කිරීම
ගොනු අංකය	BD/GH/CPMA/1/2013

නොමේරු ළදරු ඒකකයේ CX 50 අල්ට්‍රාසවුන්ඩ් පද්ධතිය හා වෛද්‍ය උපාංග පිරිසිදු කිරීම පිළිබඳව න්‍යූර්ජිවී විශේෂඥ වෛද්‍ය විසින් නිකුත් කර ඇති මාර්ගෝපදේශ මේ සමග අමුණා ඇති අතර එයට අනුව කටයුතු කරන ලෙසත් අදාල කාර්ය මණ්ඩල දැනුවත් කිරීමට පියවර ගන්නා ලෙසත් කාරුණිකව දන්වා සිටිමි.

**Director
Teaching Hospital,
Badulla.**

Care and cleaning of cx 50 ultrasound system and transducers in Pre-Term Baby Units

CLEANING AND DISINFECTANT SOLUTIONS FOR SYSTEM SURFACES

The compatibility of disinfection and cleaning solutions varies depending on item on which they are used. The products in the following table are compatible with these system surfaces;

- External plastic and painted surfaces of system and cart
- System control panel
- ECG trunk cables ,leads, and electrodes
- Monitor screens

Cleaning solutions for All surfaces	Cleaning solutions for Monitor Screens	Disinfectants for system Surfaces
Mild soap Solution	Mild soap solution	70% Isopropyl alcohol
	Cleaners designed for LCDs	
	Purified water	

Cleaning the System and ECG Equipment

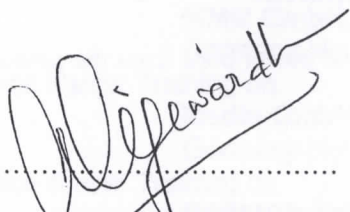
1. Before cleaning, turn off the system, disconnect the power cord from the power source, and ensure that the system brakes are locked.
2. To clean display screens;
 - a. Remove dust with a soft, lint-free cloth. Philips recommends using a microfiber cloth.
 - b. Use a liquid screen cleaner specifically designed for LCDs . Spray the liquid onto the cleaning cloth and wipe the screen clean. You can also use pre-moistened screen wipes.
 - c. Dry the screen with a soft, lint-free cloth.
3. To clean the Control panel, remove any solid matter around the keys or the controls with a cotton swab or tooth pick to ensure that solids are not pushed in to the cabinet. Wipe with a soft cloth moistened with soap and water.
4. Wipe the remaining external surfaces of the system and the cart with a soft cloth moistened with soap and water ;
 - Painted and plastic surfaces
 - ECG trunk cables, leads, and electrodes

You may use a 70% isopropyl alcohol solution for stubborn stains or inks, and then wash with soap and water.

5. Remove any residue with a cloth moistened with purified water.
6. Dry the equipment to prevent potential corrosion.

Disinfecting system surfaces and ECG Equipment

1. Before cleaning and disinfecting, turn off the system, disconnect the power cord from the power source, and ensure that the system brakes are locked.
2. Clean the system according to the procedures in above.
3. Choose the disinfection solution compatible with your system (70% alcohol) and follow the label instructions for preparation, temperature, and solution strength. If a pre-mixed solution is used, be sure to observe the solution expiration date.
4. Wipe the system surfaces with the disinfectant.
5. Dry the equipment to prevent potential corrosion.


.....
Consultant Microbiologist
MBBS, Diploma in Med. Micro, FID in Micro Micro
Consultant Microbiologist (Acting)
Provincial General Hospital
Badulla.

Copies to,

01. Consultants in charge, PBU
02. Nursing Officer in charge, PBU
03. Infection Control unit


.....

Director, TH Badulla

15/8/23

Dr. Upul Karunaratne
MBBS, MSc (Med. Admin), MD (Med. Admin)
Director
Teaching Hospital, Badulla.